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## Information about the usage of the product

### One time face mask Type IIR Mouth-Nose Protective Mask

**Purpose:**

The mask is suitable for use in all medical and non-medical settings. It can be used by medical personnel.

**Typical applications:**

OR areas, nursing services, laboratories, nursing homes, utilities, pharmacies, food industry, etc.

The masks are not suitable for wearing during competitive sports or permanent heavy physical work.

The masks are single-use face masks and should be disposed of properly after use.

The masks are NOT washable and NOT reusable.

**Testing:**

The mask has been tested by Hohenstein Laboratories GmbH & Co. KG according to the standard DIN EN14683:2019-10 and confirmed for the field of application (Type IIR). Test end numbers: 0843 / 0848 - July 2020

**Putting on the mask:**

Put the mask on with the colored fleece on the outside and the nose clip on top by pulling the folds apart in the middle. Clip the ear bands behind the ears on both sides. The mask must reach to the base of the nose under the eyes and is adjusted to the shape of the nose with the nose clip. It thus completely covers the nose and mouth.

Click here for the declaration of conformity:

For further questions please contact us: [info@ect-solutions.de](mailto:info@ect-solutions.de)

Place of production: Großostheim (Bavaria), Germany  
Storage temperature at 15°C - 30°C / 3 years from date of production

Registered office of the company: Philipp-Kachel-Str. 42a - 63911 Klingenberg / Germany  
Email: [info@ect-productions.de](mailto:info@ect-productions.de)